

Quick Pressure Monitor Set **Sterile Procedures**

The Stryker Intra-Compartmental Pressure Monitor System is a convenient, self-contained, reliable unit for an immediate or continuous reading of compartment pressure.

Instructions

1. First you must turn unit on.
2. Remove contents of disposable pouch.
3. Place needle firmly on the tapered chamber stem.
4. Remove cap on pre-filled syringe and twist the syringe against remaining chamber stem without contaminating the fluid.

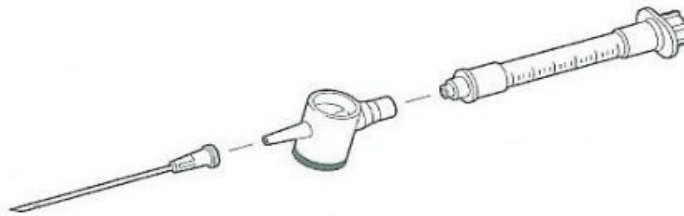


Figure 1

5. Open the monitor's cover and place slot in well, gently pushing it until disposable settles in.

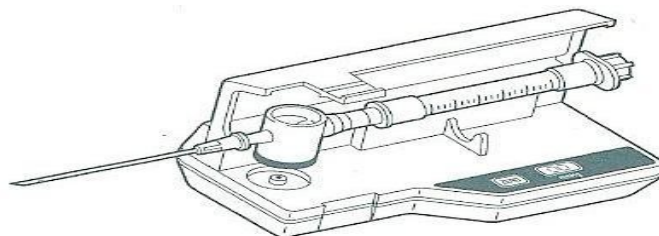


Figure 2

6. Snap cover closed without forcing it.
7. Pull cap off syringe and attach needle rod.
8. Slowly force fluid through the disposable to flush out air without allowing saline to pour down the needle.
9. Fix the intended angle of insertion of the needle into the skin. Press the zero button and wait 2 seconds. The display should read 00.

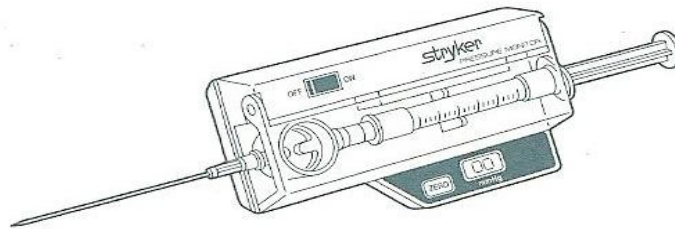


Figure 3

10. Insert needle into body. Slowly inject less than 3/10cc of saline into the compartment.
11. Wait for the display to reach equilibrium before reading pressure.
12. For additional measurements, make sure to reset unit and repeat steps 9-12.